AMORELIE
Joy

Glow
Liebeskugeln

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Was ich bin & was ich mache

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WHAT I AM & WHAT I DO Love balls to strengthen the pelvic floor muscles - can result in more intense orgasms.

LET'S DO IT... Rub a little water-based lubricant onto the love balls and insert as far as the retrieval strap – the same as you would a tampon. Intentionally tense the pelvic floor muscles to keep the balls inside you, and keep moving. How about a touch of light yoga? Wear once or twice per day, for around 15 minutes, but not more than 4 hours a day. Take a day's break if you experience aching muscles, and only continue training when you no longer feel any pain.

... AND USE IT PROPERLY! ♥ Use only water-based lubricant. ♥ Massage oils are unfortunately a no-go – sorry! ♥ Clean the toy carefully with water and mild soap before and after every use. ♥ Tip for clean freaks: use toy cleaner! ♥ Store the toy in a secure and dry location ♥ Exposure to direct sunlight or extreme temperatures is a big no no.

YOU WANT TO KNOW EXACTLY, RIGHT? Technical specifications ♥ Geruchsneutralen Seife Made from certified silicone. ♥ Is waterproof and loves going for a dive ♥ Length: 17.1 cm, Width 2.9 cm, Height: 2.9 cm, Diameter: 2.9 cm, Total weight: 75 g ♥ Insertable length: 9 cm

PROTECTING YOUR HEALTH • In the following situations, consult your doctor before using the toy: - Illness or adhesions, - Incontinence or prolapse, - Pregnancy and/or - Use of a contraceptive coil. • Do not wear the toy in the following situations: - if you are suffering from swollen, irritable or inflamed skin. - in case of vaginal infection, - during intercourse, - during menstruation, - during the first 6 weeks after childbirth, - following surgery in the pelvic area, - if any parts become damaged and/or - if you have an allergic reaction to the material or if you are aware of an existing allergy to the material.

• If you feel any pain and/or discomfort during use, you should stop using the product immediately and refrain from any further use. • To avoid illness and infection, do not share the toy with others. • Do not clean using products that contain alcohol, petrol, acetone, or other corrosive substances. Not suitable for washing machines or dishwashers. • Store the product out of the reach of babies, children, pets and animals. • Only suitable for vaginal use. The toy is only to be used as intended and as directed within the scope of its functional description. Not suitable for medical use.

PROTECT THE ENVIRONMENT This product can be disposed of in your household waste without causing harm to the environment.

WARRANTY STUFF We guarantee that this product will remain free from material and production defects for a period of two years from the original date of purchase. This guarantee does not cover product damage or defects that are due to an accident, misuse, dirt, water, tampering, unsuitable use, normal wear and tear, or repair that is not performed by an authorised service office. In the event of a warranty claim, we will replace the product for you free of charge. Your statutory rights - defect-related rights pursuant to §§ 437 ff. BGB - are not affected by this warranty.

S.O.S. – MY TOY WON'T DO WHAT I WANT IT TO! If you require assistance, you can find answers to many questions, along with our contact form, in the Help Centre: support.amorelie.com EQOM Purchasing B.V. Phoenixweg 6. 9641 KS Veendam, The Netherlands. All rights reserved. Made in China. Curious as to which other toys are available from AMORELIE? Visit our shop: amorelie.com

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